

For Safe & Easy Measurements of Urine Specific Gravity!!



DIGITAL HAND-HELD "PEN"
URINE S.G. REFRACTOMETER

Pen - Wrestling

Cat.No. 3749

Hydration

Preferred method of testing to meet
NFHS weight certification guidelines.

The PEN-Wrestling provides accurate,
worry-free testing of Urine Specific
Gravity to determine acceptable
hydration levels.



Features

- Calibration with water only
- Measurement in 2 seconds
- Extremely water resistant (IP65)
- Light & Compact, 70g
- Ergonomically designed for easy, one-handed operation
- External Light Interference (ELI)
- Convenient storage case
- Automatic Temperature Compensation (ATC)

Specifications

Measurement range	: Urine S.G. 1.0000 to 1.0600
Resolution	: Urine S.G. 0.0001
Measurement accuracy	: Urine S.G. ±0.0010
Measurement temperature	: 10 to 40°C (ATC)
Measurement time	: 2 seconds
Power supply	: AAA alkaline battery × 1
Battery life	: Approx. 8,000 measurement (When an alkaline battery is used)
International Protection Class	: IP65 Water resistant (Prism head:IP67)
Dimensions and weight	: 160(W)×38(D)×18(H)mm, 70g

The PEN type refractometer is extremely easy to use!

Fast measurement results: safe and hygienic.



Measurement Methods

Dip the tip into the sample and press the START key OR press the START key (and within 10 seconds) touch the tip into the sample. The measurement value will be displayed in 2 seconds.

*Small Sample Volumes

Quick & Easy Cleanup!



All ATAGO refractometers are designed and manufactured in Japan.

CE HACCP GMP GLP ATAGO products comply with HACCP, GMP, and GLP system standards.

ATAGO CO., LTD.

Headquarters: The Front Tower Shiba Koen, 23rd Floor
2-6-3 Shiba-koen, Minato-ku, Tokyo 105-0011, Japan
TEL : 81-3-3431-1943 FAX : 81-3-3431-1945
overseas@atago.net http://www.atago.net/

ATAGO U.S.A., Inc.

TEL : 1-425-637-2107 customerservice@atago-usa.com

ATAGO INDIA Instruments Pvt. Ltd.

TEL : 91-22-2833-8038 / 8076 customerservice@atago-india.com

ATAGO (THAILAND) Co., Ltd.

TEL : 662-982-8718-9 customerservice@atago-thailand.com

ATAGO BRASIL Ltda.

TEL : 55 16 3916-6000 customerservice@atago-brasil.com

ATAGO ITALIA s.r.l.

TEL : 39 2 36557267 customerservice@atago-italia.com

ATAGO CHINA Guangzhou Co., Ltd.

TEL : 86 20-38106065 info@atago-china.com



* Specifications and appearance are subject to change without notice.

novatech
INTERNATIONAL

sales@novatech-usa.com

www.novatech-usa.com

Tel: (866) 433-6682

Fax: (866) 433-6684

Tel: (281) 359-8538

Fax: (281) 359-0084